SMART JECT 사용설명서 | User Manual

SMART JECT

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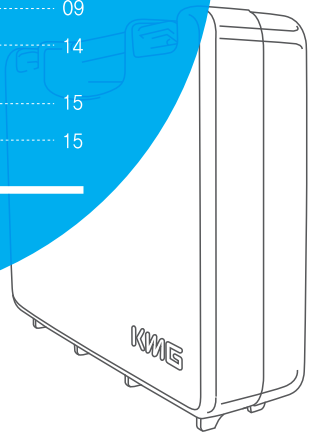
Thank you very much for purchasing this motor-powered injector, **SMART JECT**.
 Be sure to get fully familiar with all guidelines and instructions in the User Manual
 for correct use and proper maintenance. Keep the enclosed product warranty in a safe place.

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This instrument is intended for injection of medications such as local anesthetics; it consists of a main body and a battery charger, designed to administer an anesthetic agent at a certain flow. The device can control injection quantity at a 1/16, 1/8, 1/4, 1/2, or FULL ampule and also has an aspiration function. Furthermore, the injection rate can be regulated at high and low speeds.

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The device can hold Lidocaine ampules and a 4ml general-purpose syringe, enabling administration of various medications.

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The device can control the setting of injection quantity and the rate of injection flow (high/low speed).

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The device comes to a full stop when its sensor detects any pressure that goes higher than the pre-determined level.

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The device has an LCD panel where the user can identify battery level, function settings, and the progress of all tasks.

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The device has a speaker by which the user can confirm the operation of tasks.

- 전원 ON 알림음 (Power-On signal) / - 전원 OFF 알림음 (Power-Off signal)
- 후진(aspiration) 알림음 : 모터가 후진을 때 발생하는 음
Backward motion alarm (for aspiration) : the sound that goes off at backward movement of the motor.
- 주입완료음 : 앰플의 전체를 소모하였을 때 발생하는 음
Ampule completion sound : the sound goes off when an ampule runs out completely.
- 위치알림음 : 앰플 잔량이 분할 지역에 위치하였을 때 발생하는 음
Position indication sound : the sound goes off when the remaining quantity of an ampule reaches a marked nick.

- 앰플의 절대위치 1/8단계에 도달하였을 때
When an absolute consumption level reaches to the 1/8 mark(1/8)
 - 앰플의 절대위치 1/4단계에 도달하였을 때
When an absolute consumption level reaches to the first quarter mark(1/4)
 - 앰플의 절대위치 2/4단계에 도달하였을 때
When an absolute consumption level reaches to the second quarter mark(2/4)
 - 앰플의 절대위치 3/4단계에 도달하였을 때
When an absolute consumption level reaches to the third quarter mark(3/4)
-
- 저속음 : 모터가 저속으로 동작하고 있을 때
Low-speed sound : the sound goes off when the motor runs at the low speed,
 - 고속음 : 모터가 고속으로 동작하고 있을 때
High-speed sound : the sound goes off when the motor runs at the high speed,
 - 버튼 키 소리 : 키를 누를 때 발생하는 음
Button sound : the sound goes off when buttons are pressed,

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When the cartridge is mounted and power is allowed, the motor starts running, The cylinder begins to move, pushing the anesthetic drug ampule from the cartridge and passing the liquid through the needle.

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Do not use the product in a humid place like a bathroom.

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Check the device for operation before it is put to use.

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Check buttons for accurate operation beforehand.

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Do not bend, distort, or knot cables, for they may become overheated during use and cause danger.

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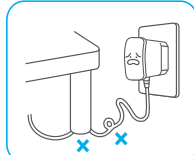
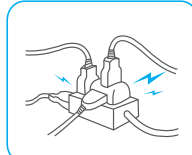
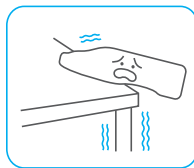
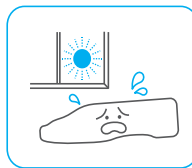
Do not step on the plug or give too much pressure to it.

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Do not use something inflammable near the product.

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When the product develops a problem, stop using the product and turn it off immediately.

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When the product has a problem, contact the store or the maker for repair.

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Do not spray water directly or use such chemicals as thinners and benzene, when cleaning the product.

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Main body: is the major component of the product where drug ampoules are installed, It regulates the rate and speed of the drug in manners as the user intends,

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Charger support: is a structure that holds the main body securely and helps charge the battery,

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Battery: is installed in the main body to power the product, It is rechargeable,

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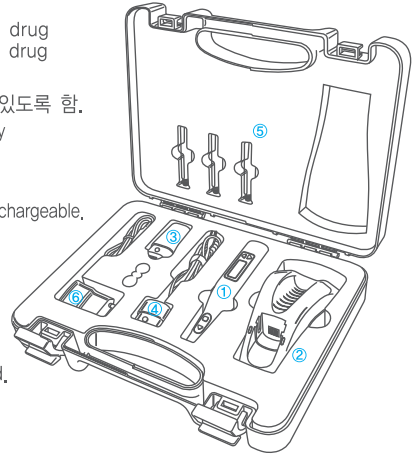
Wire jack: is a cable to supply power to the product by using a battery recharged,

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Cartridge: is where drug ampoules and needles are mounted,

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AC adapter: is a device for charging the battery,

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
| Description of exterior parts- Charger support

- 1 충전완료 LED (청색 등) | LED for end of charging (Green LED)
- 2 충전중 LED (적색 등) | LED for charging in progress (Red LED)
- 3 배터리 커넥터 | LED for charging in progress (Red LED)
- 4 유선잭 연결 커넥터 | Wire jack connector
- 5 아답터 커넥터 | Adaptor connector
- 6 본체 거치대 | Main body support

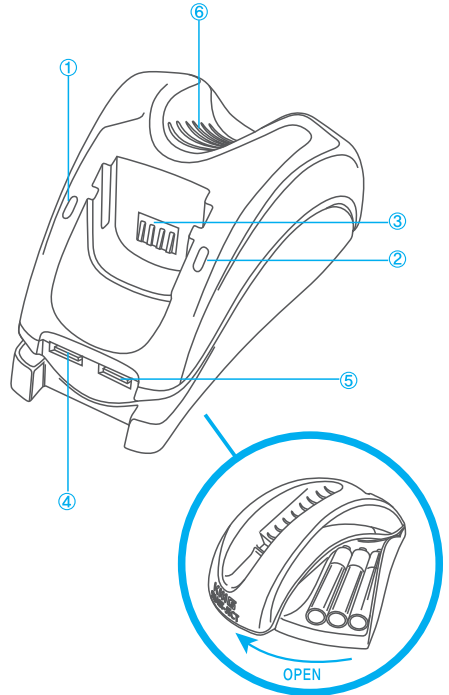- 1 충전이 완료되면 녹색 등이 점등
This LED indicates the end of charging process, when it turns green
- 2 충전중이면 빨간색 등이 점등
This LED indicates charging in progress, when it is lighted red
- 3 충전할 배터리를 연결하는 커넥터
The connector is for connection of a battery to be charged
- 4 유선잭을 연결하는 접속 커넥터
The connector is for connection of the wire jack
- 5 5V/1A 아답터 접속 커넥터
The connector is for connection of a 5V/1A adaptor
- 6 본체를 거치하는 장소
The support is for mounting the main body

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
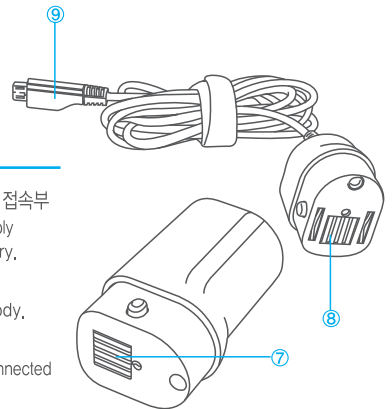
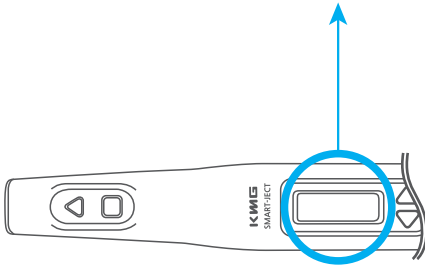
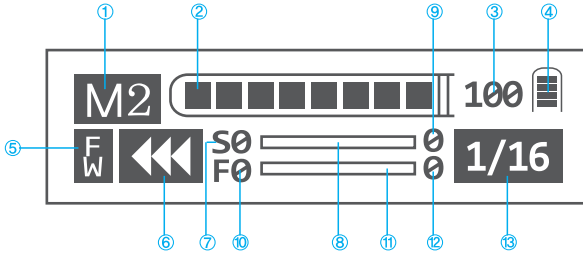
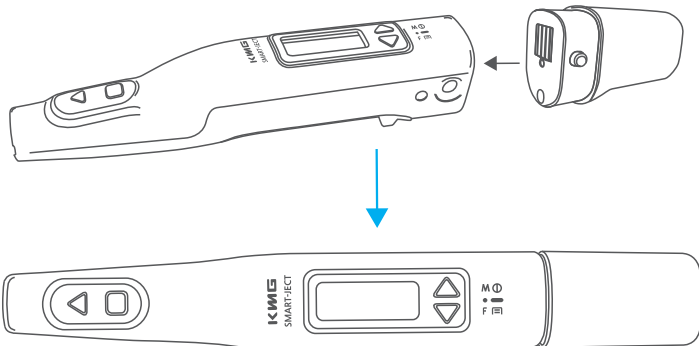
Installation of battery: A completely charged one should be connected to the main body.

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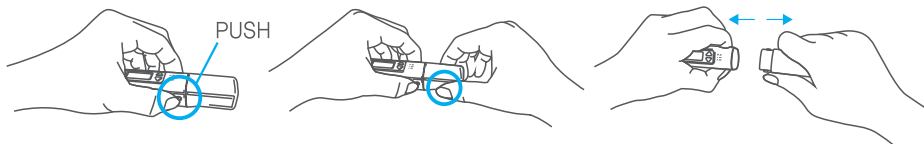
Disassembly of battery: Press the protrusions on both sides of the battery with one hand and pull to disassemble it with the other hand.

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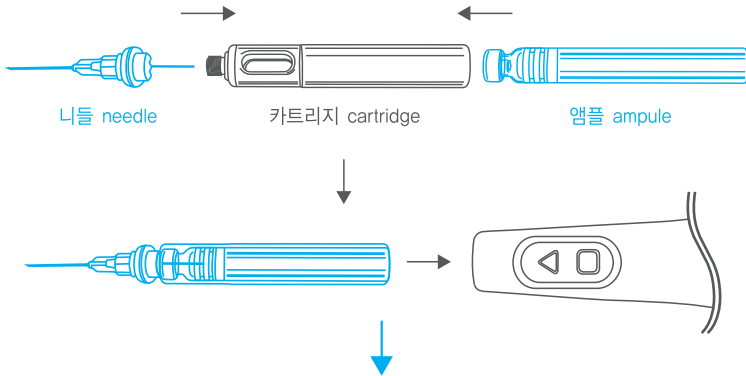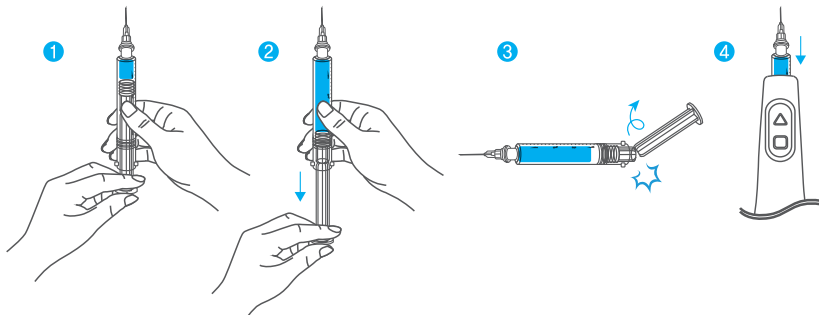
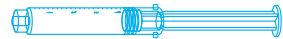
Installation of cartridge: The cartridge is first installed with an ampule and a needle and inserted into the main body. Turn 90 degrees to the right (clockwise) to mount it; turn 90 degrees to the left (counterclockwise) to dismount it.

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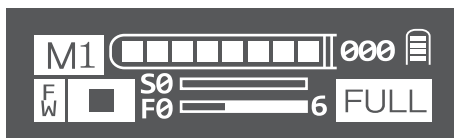
Installation of 4ml syringe-use cartridge: Tear off the plastic like ③ when the bottom of a syringe is pulled with hand to let the drug fill the tube like ②, when the syringe can be mounted into the main body.

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
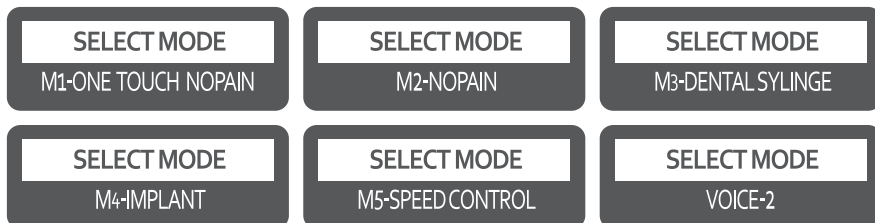
3. 주입량 선택 (M1~M5 동일) Injection quantity selection (M1~M5)

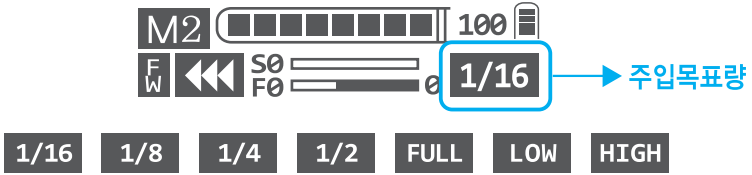
- ① S4버튼을 짧게 눌러서 주입량을 선택한다.
Make a short press of the S4 button to select a desired injection rate.
- ② 주입량은 1/16, 1/8, 1/4, 1/2, FULL, LOW, HIGH로 선택된다.
An injection rate and a speed can be chosen from 1/16, 1/8, 1/4, 1/2 and FULL and LOW and HIGH.

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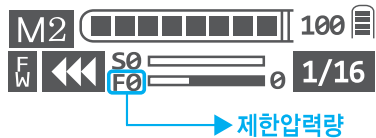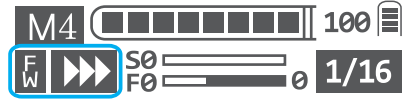
If the aspiration mode is required, press the S2 button and motor goes backward, release it and the device will stop.

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Voice control: The mode setting window appears when the S3 button is pressed for 2 seconds. Move to the voice setting window using the S3 button. Press the S4 button to obtain your desired volume level from VOICE OFF~2. Press the S3 button to exit the voice control mode. However, be sure to set your desired use mode after the voice level is determined.

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Warranty period: for main body, valid for one year from the date of purchase; for battery, for 6 months

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Any failure resulting from or in connection with faults, mistakes, or errors attributable to the user shall be subject to paid service.

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When a problem occurs in connection with negligent or careless handling, or changes, modifications, or alterations of the product by the user,

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When a problem or damage arises after installation from any other factors attributable to the user including drops during transfer from one place to another,

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When a problem takes place in relation to use of the product for any purposes other than originally intended,

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Freight cost incurred from transportation for repair and any other consequential expenses there of shall be borne by the customer.

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