Lighting on your hand
Portable Lighting System

Helio

User manual

Eng

Kor

1. Introduction	02
- Contents	
- Features	
- Description	
2. Safety Information	03
- Symbols	
- Notes and reference	
- Installation and Connection	
- Operation	
- Maintenance	
3. List of Components	07
4. Parts and Functions	08
5. Installation	09
6. How to use	12
7. Maintenance	20
8. Trouble Shooting	21
9. Specification	22
10. Accessories	23
11. Warranty	23
12. Quality Assurance	24

1. Introduction

Thank you for purchasing the device. The instructions described below have been designed to provide the necessary information for the successful operation, daily maintenance and safety for the use of this device.

Features

- Lighting anywhere
- Autoclaveable
- Cordless type
- Practical design
- Big capacity battery
- Smart system

Description

Portable lighting system for oral cavity

2. Safety information

It is important to understand the following information to use this product safely.

Symbols

Indicates a potentially hazardous situation which, if not avoided, may result in serious injury or death.

Indicates a potentially hazardous situation which, if not avoided, may result in inor/moderate injury or in damage of property.

Notes and References

Indicates that potential malfunctions may occur if the description is ignored.

Supplementary information to facilitate better operation.

Installation and Connection

Warning

- The adapter should be disconnected from the charger before connecting the power cable to prevent an electric shock.
- Keep away from sources of ignition to avoid fire or explosion.
- Protect the device from any liquid to avoid electric shock or functional error.
- Place the charger on a flat and stable surface. Do not drop device and charger to avoid internal defect that may cause overheating or fire.
- Do not place the electrical cord near a gas burner or sources of heat. Should the cord catch on fire, replace with the new one to avoid short-circuit.

Caution

- Do not use or leave the unit in areas of high temperature or exposed to direct sunlight, internal circuit defect may cause overheat or fire.
- Securely connect the dust-free power plug to the outlet.
- Do not touch the terminal areas of charger or battery with fingers or conductive objects, such as metal wire or safety-pin. It may electric shock or fire.
- Do not use other charger. It may cause fire or device malfunction.

Warning

- Do not plug the power cord with wet hands. This may result in an electric shock.
- In case of fluid leak from battery or partial discoloration / deformation of the device, immediately stop use.
- Place the battery on correct direction when charging. Wrong directed placing may cause malfunction, explosion, or fire.
- In case partial discoloration of the device, immediately disconnect the power plug and repair the device.
- In case battery fluid contacts the eye, immediately flush with copious amount of clean water and get medical attention.
- Do not use this device on patients with a history of cataract, retinal damage or allergic reaction to light.
- Do not use this device on a person with a pacemaker as it may cause malfunction of pacemaker.
- Do not put the battery on the charger upside down. This may cause an explosion or fire.

Caution

- Disconnect the power cord by grasping the power plug.
This may cause injuries or burns.
- Avoid damage or use of unnecessary force on the power cord.
This may cause an electric shock or fire.
- Should battery fluid contact the skin or clothes, immediately wash it off with copious amount water to avoid skin damage.
- Take the battery out of from device, if it won't be used for long time.
- Connect the power plug only when battery is being charged. Insulation degradation may cause electric shock or fire.
- Do not use this product for any purposes other than dental use.

Maintenance

Warning

- Do not autoclave the battery. It may cause explosion or fire.
- Use only the battery supplied to avoid fluid leak or explosion.
- Never disassemble or overhaul the product. Unusual performance may cause injuries, electric shock, or fire.
- Ensure that charger and battery are always separated from the device prior to cleaning


Caution


- If the device doesn't work normally, remove the battery from the device and reinsert it.
- User must be in charge of operation, maintenance and check.

3. List of components


Please check the contents described below when removing the contents from the original packaging.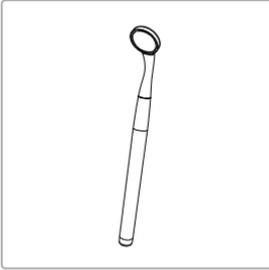


Handpiece (Head & Body)

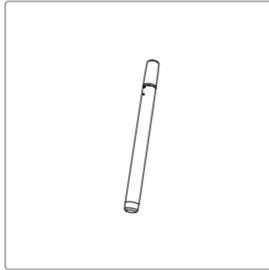
Battery

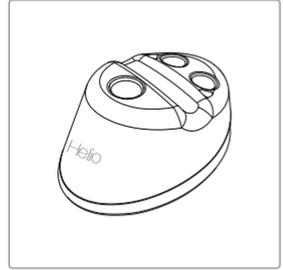
Charger

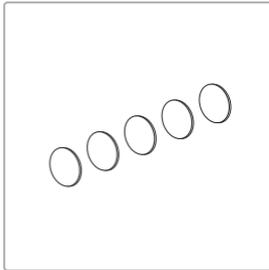Mirror (5 pcs)

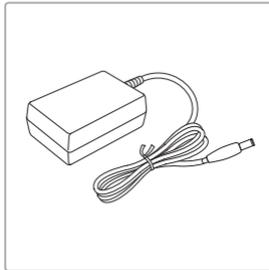
AC adapter

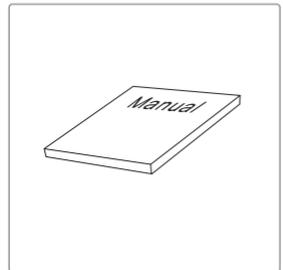Manual

4. Parts and functions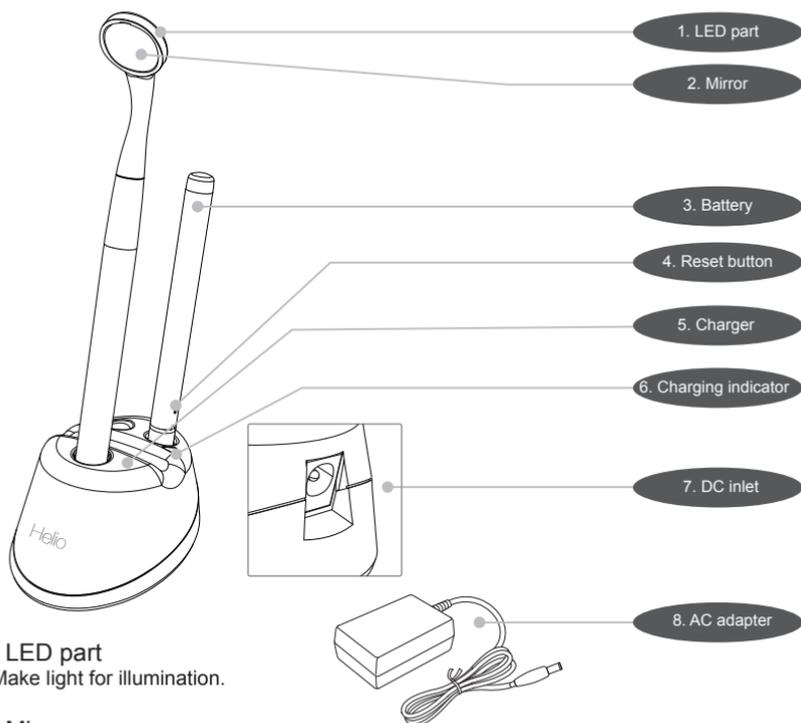

1. LED part

- Make light for illumination.

2. Mirror

- Interchangeable mirror. Magnetic makes holding the mirror.

3. Battery

- Included rechargeable battery and electric circuit.

4. Reset button

- When device doesn't work normally, push the reset button gently.

5. Charger

- Recharge the battery.

- Can recharge 2 batteries at one time.

6. Charging slot

- There are 2 charging slot.

7. Charging indicator

- Power connect: Both LED are blinked

- On charging: Red LED is shown

- Full charging: Green LED is shown

8. DC inlet


9. AC adapter

- Input: 100~240 V

- Output: 5V


5. Installation

1. Connection between AC adapter and charger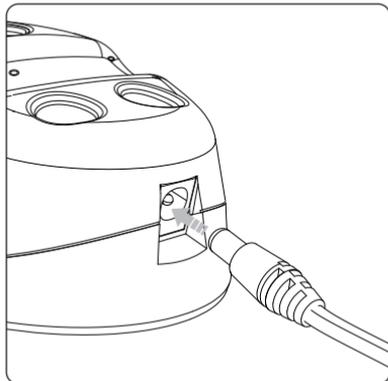

- Securely insert the connector of the AC adapter into the inlet at the back of the charger.

- The AC adapter is able to use with a power outlet of 100V~240V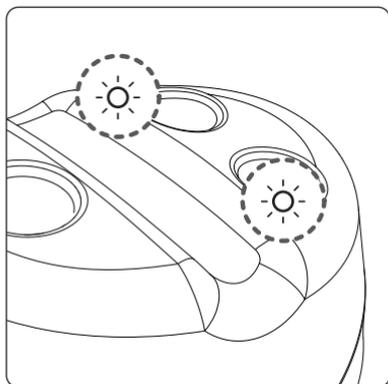

- When the power plug is connected green lights flash instantly which is not a malfunction.

It is alarm sign to check the charger condition.


Warning

Use only the original AC adaptor supplied.

5. Installation

2. Connect mirror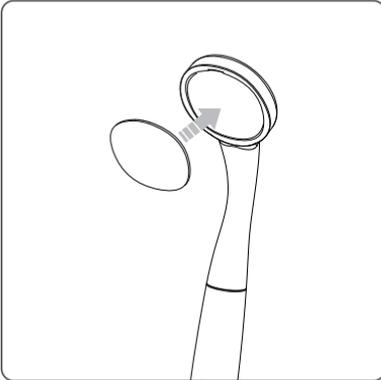

- Connect the mirror as shown.
Magnetic inside of device hold the mirror.


Warning


If the mirror connection is not good, it would be detached on operating.
Connect mirror, and check it is connected well.

5. Installation

3. Inserting battery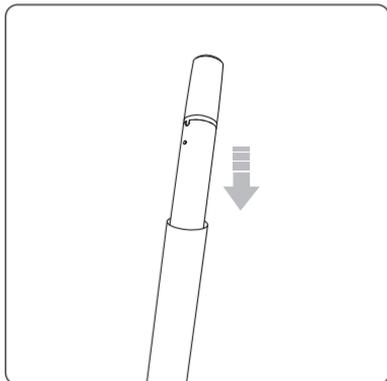

- Carefully insert the battery into the handpiece as directed by the arrow certainly. Avoid forcing the battery.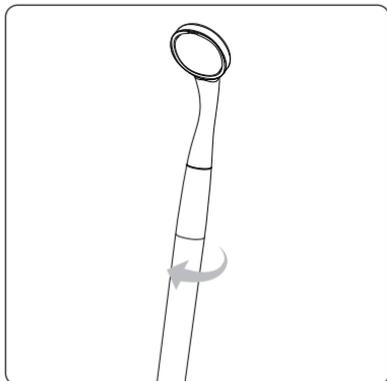

- Spin the body as picture certainly. After connection the LED light turn on.


Note

- The device works when the battery is connected.
- To turn off LED light, disconnect the battery, or put the device on the stable site, or shake the device 2 times for vertical direction.


6. How to use

1. Activating device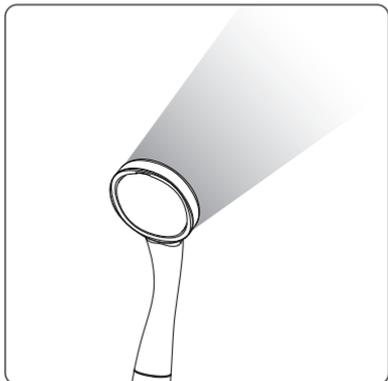


- When battery is inserted, the device is automatically switched on.

2. Brightness changing

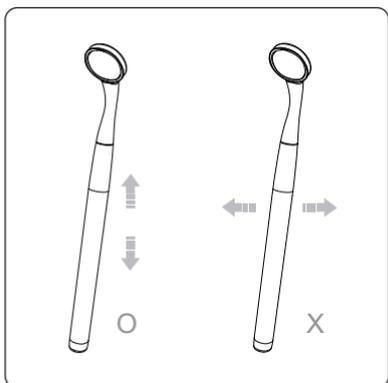
- Shake the device by hand for vertical direction.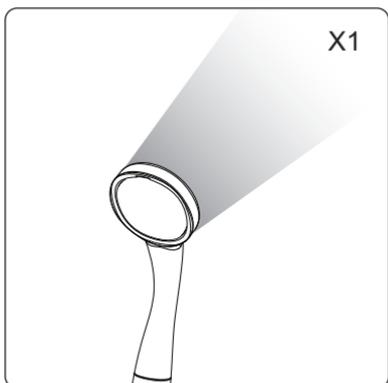


- Low brightness with one time blinking.

6. How to use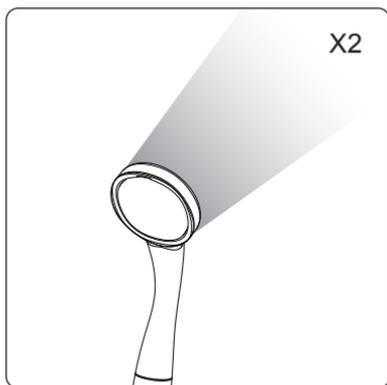

- High brightness with two times blinking.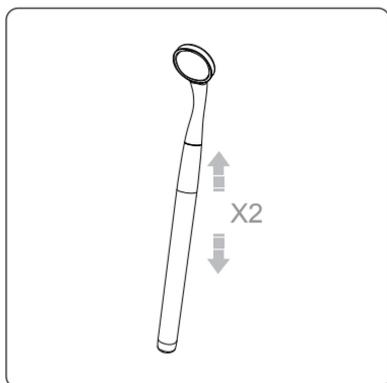

- Shake the device by hand for vertically direction 2 times. Then the LED light turn off.


Reference


To avoid changing light power when remove waterdrops on the mirror surface, the light power is not changed when shake with upside down direction even shake the device vertical direction.

6. How to use


3. Turn off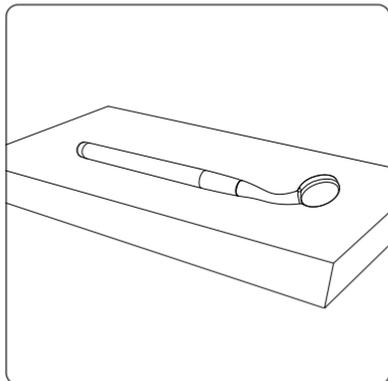

- Put the device on stable site. After few seconds, the device is turned off automatically.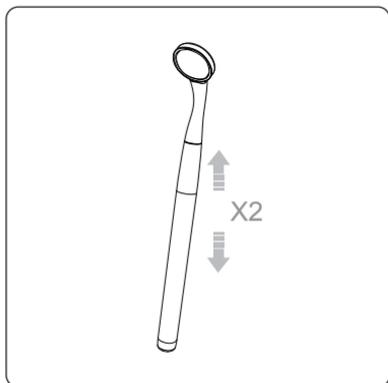

- Shake the device by hand for vertical direction 2 times.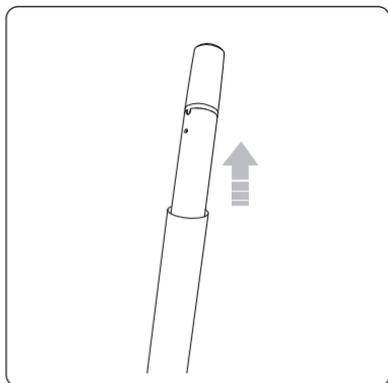

- Disconnect the battery.


Note


The device is very sensitive for shaking. The device would not tune off even small shaking.

6. How to use

4. Battery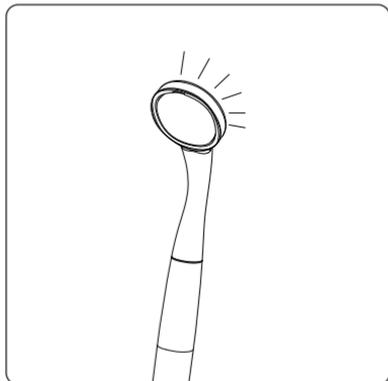

- When the battery needs charging, the LED light is blinking by itself.

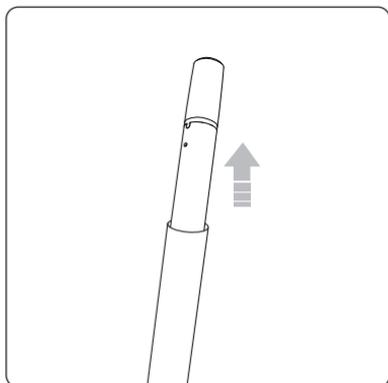- Separate the battery and recharge it.

Note


If the battery voltage is too low, the LED light is turned off.

6. How to use

5. Recharging

- Connect the battery on the charging slot as shown smoothly.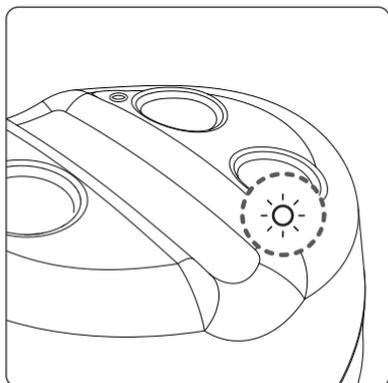

- Red light is shown on charging.
- Blue light is shown on charging is done.

Note

In case of the battery voltage is too low, the charging indicator could blink.
It is normal situation.


Reference


There are two slots for charging the battery. If you have additional battery, you can charge two batteries at one time. The charging indicator works individually.

6. How to use

6. Changing mirror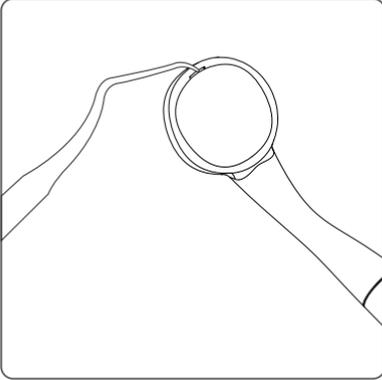


- If the mirror get scratch or need cleaning, need to detach the mirror.
- Use kind of pincette to remove the mirror as shown.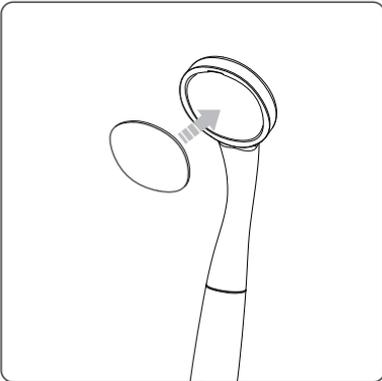


- Connect the mirror as shown. Magnetic inside of device hold the mirror.

6. How to use

7. Storage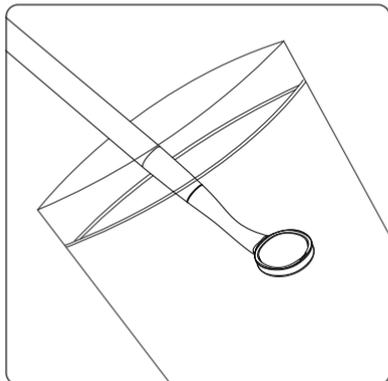

- Separate the battery, and clean the handpiece.
- Put the handpiece into sterilization pouch and sealing it.

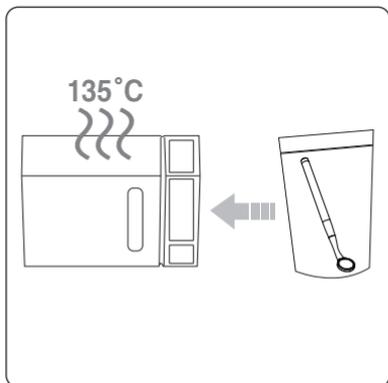
- Autoclave the device.

- The battery is not autoclaveable. If it is on autoclave, it would be exploded.
- Check it when autoclave the device that the battery is included or not.

6. How to use

8. Example for use

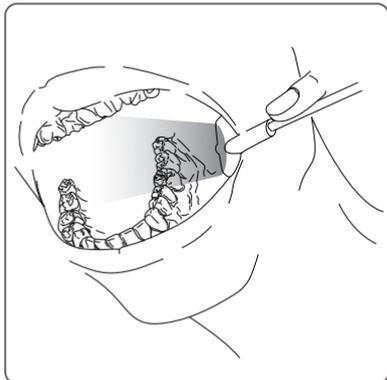
- Use as a retractor like as shown.
- For implant, surgical operating, or etc.

Note

- When the device works up-normally for example the light could blinks, lower illumination, please reset the device.
- Push the reset button on battery with kinds of pin as shown.

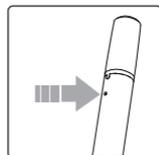7. Maintenance

This device is used operation generally. So cleaning and sterilization is very important. Please refer the follow information when you cleaning, sterilization and maintenance.

Cleaning

- Avoid forcing the device and charger to prevent electric shock or malfunction. Ensure that the charger and battery are always separated when cleaning.
- Clean the handpiece and charger gently with a clean, soft, lint-free cloth. Avoid using solvents containing oil, ethanol, or acetone.
- Remove contamination or dirty with soft pointed tools or plastic spatula.
- Battery is not warterproof. Careful when clean the battery.

Sterilization

- Envelop the handpice into the sterilization pouch.
- Put the handpiece in the autoclave machine.
- If the battery is sterilized in autoclave machine, it would be explode.
Check the battery is separated when autoclave the handpiece.
- The temperature should under 135°C.
If the temperature is exceed over 135°C, the device would get damage or malfunction.

Warning

- The battery is not autoclaveable. If it is on autoclave, it would be exploded.
- Check it when autoclave the device that the battery is included or not.
- The autoclave temperature should under 135°C. If the temperature is exceed over 135°C, the device would get damage or malfunction.

Maintenance and replacement of the battery

- Exposure to high room temperature may shorten the battery life. room temperature ranges between 15 to 25°C normal temperature.
- It is recommended for the rechargeable battery to be charged immediately before use to maximize the efficiency.
- Remove the battery from the handpiece, if not being used for long time.
- If the running time declines with a full-charged battery, replace the battery before use.

8. Trouble shooting

If the device is not operating normally, please refer to the following points.

Problem	Cause and solution
LED light is not turned on.	<p>If the battery voltage is extremely low, the device would be not work. Please charge the battery.</p> <p>If the autoclave temperature is higher then 135°C, the device would be get damage. Please check the autoclave temperature is correct or not.</p>
Charging indicator is not work.	<p>Please check the connection of battery and charger.</p> <p>Please check the AC adapter is connected well.</p>
Power off while in operation	<p>If the battery voltage is extremely low, the device would be turned off. Please charge the battery,</p> <p>Please check the connection between battery and handpiece.</p> <p>Reset the device.</p>
Not able to charge	<p>If the power cord is not correctly connected to the charger, or the battery is not properly placed on the charger.</p> <p>Clean the charger or the charging terminal.</p>
When light is poor	<p>If the battery voltage is extremely low, the light would be poor. Please charge the battery.</p> <p>If the battery life is done, the battery voltage is lower then normal condition. Please replace the battery to new one.</p> <p>Reset the device.</p>

* If the trouble is not fixed after following solution, please contact your local dealer or manufacturer.

9. Specification

Equipment Class

Model:	LS-HMT30
Safety:	IEC60601-1
EMC (Electro-Magnetic Compliance):	IEC60601-2
Protection from electric shock:	Type B 
Protection from ingress of liquids:	IPX0 (ordinary equipment)
Operation:	Intended for continuous operation

Operating environment

Ambient temperature:	15°C to 35°C
Relative humidity:	10% to 80%
Atmospheric pressure:	500 hPa to 1060 hPa

Transport and Storage Environment

Ambient temperature:	-20°C to 60°C
Relative humidity:	10% R.H. to 85% R.H.
Atmospheric pressure:	500 hPa to 1060 hPa

Additional Information

Handpiece	Dimensions	Approx. 190 x 25x 25 mm
	Weight	Approx. 20 g
Charger	Dimensions	Approx. 105 x 68 x 55 mm
	Weight	Approx. 100 g
Battery	Type	Lithium-ion battery
	Nominal voltage	DC 3.7V
	Dimensions	Approx. 125 x 10 x 10 mm
	Weight	Approx. 20 g
Mirror	Dimensions	Approx. 20.5 mm
AC adapter	Input	100 ~ 240 V / 50~60 Hz
	Output	5V

* this specification is subject to change without prior notice.

10. Accessories

- Plus Pack (Handpiece x 2 EA, Battery x 1 EA, Mirror x 1 pack)
- Handpiece
- Battery
- Mirror (1 pack = 5 pcs)
- AC adapter

* If need to purchase accessory, please contact local supplier or manufacturer.

11. Warranty

This device is only shipped after strict inspection. Should the product incur any problems under normal use conditions within one year from the date of purchase, the repair will be free of charge. However, repair of consumables (Mirror, Battery, and etc.) are not covered by the warranty. For more information, refer to the attached Quality Assurance.

 Gruneworld GmbH
Georg-Weiss-Weg 9, 53125 Bonn, Germany
TEL: +49(0)288/7077695 FAX: +49(0)288/7077696
Website: www.gooddrs.de

 Good Doctors Co., Ltd.
283, Bupyeong-daero, Bupyeong-gu, Incheon, Korea
(#208, B-dong, Woolim Lions Valley) (Zip code : 403-911)
Tel : + 82 32 424 6325 Fax : + 82 32 424 6326
E-mail : info@gooddrs.com Website : www.gooddrs.com

Good Doctors Good Doctors Good Doctors Good Doctors Good Doctors

Quality Assurance

We guarantee the quality of the product as follows

Name of product :

Name of model :

Customer information

- Name of dental clinic :

- Name :

Distributor

- Seller :

- Address :

- Telephone :

Assurance period

- Date of purchase : from _____ , 20_____

Product is for 1 year except accessories.

< Assurance details >

1. If there is any defect of manufacturer or product has any fault itself with in the assurance period, it will be replaced with the same product.

2. Below cases are not effective for the free remuneration.

- Malfunction and damage according to user's carelessness.

- Malfunction and damage according to natural disaster such as fire, earthquake and flood etc.

- Malfunction and damage according to user's intention or accident.

- Malfunction and damage according to using incompatible product.

3. The manufacturer does not accept any liability resulting misuse.

4. The compensation method which compensate it based on consumer law.

Good Doctors Co.,Ltd.

Address : #208, B-Dong, Bupyeong Woolim Lions Valley, Cheongcheon 2-dong, Bupyeong-gu, Incheon, Korea (Zip code : 409-911)

Tel : +82 32 424 6325 Fax : +82 32 424 6326 E-mail : info@gooddrs.com Website : www.gooddrs.com

Good Doctors Good Doctors Good Doctors Good Doctors Good Doctors

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