# ventura mix 2

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## EN INSTRUCTIONS FOR USE

### TECHNICAL SPECIFICATIONS

Vibrating frequency : from 3,900 to 4,500 ± 3 % C.P.M.  
Mixing time : Freely adjustable from 1 to 99 seconds.  
Motor power consumption : 70 Watts.  
Power supply : 220/240 V / 50 Hz.  
(Upon request : 115 V / 60 Hz).  
Dimensions : Width : 215 mm - Height : 145 mm - Depth : 220 mm.  
(Without cable).  
Net weight : 3.70 Kg.  
Protective cover over capsule support arms.

### INSTRUCTIONS FOR USE

1. Connect the unit to a 220 Volts / 50 Hz power outlet fitted with earth terminal.

2. To turn on the unit, press the ON - OFF lateral switch.

3. Forks speed-oscillations change :  
Press SET button for about 10 seconds. Timer will display the set programme number.  
Using buttons, the programme will be changed as follows.  
To set the chosen programme, press SET button once.

\*Programme 1 : 3.900 C.P.M. \* Programme 5 : 4.300 C.P.M.  
\*Programme 2 : 4.000 C.P.M. \* Programme 6 : 4.350 C.P.M.  
\*Programme 3 : 4.100 C.P.M. \* Programme 7 : 4.400 C.P.M.  
\*Programme 4 : 4.200 C.P.M. \* Programme 8 : 4.500 C.P.M.

4. Choice of the mixing time from 1 to 99 seconds  
Press SET button once for adjustment in steps of 10 seconds each.  
Press increase the mixing time by 10 seconds at a time.  
Press decrease the mixing time by 10 seconds at a time.  
Press a second time for adjustment in steps of 1 second each.  
Press increase the mixing time by 1 second at a time.  
Press decrease the mixing time by 1 second at a time.  
Press a third time to save the programmed mixing time, the display will stop flashing.

5. Place the capsule in the forks and close the safety lid carefully.

6. Press the START-STOP button to make the mixing/vibration process of the capsule start. If you want to stop the process, press the START-STOP button at any time, or lift the safety lid.

7. VENTURA MIX 2 is ready to be used. For safety reasons, the device only operates with the safety lid completely closed.

NOTICE : When the device is cold, we suggest doing a mixing process without capsule. At the end of the mixing/vibration period, the amalgamator remains programmed to the mixing time and programme adjusted last (even if the device is switched off or disconnected). By simply pressing the START- STOP button, the mixing process will be repeated.

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