


# Canal-Seek™

## Canal Orifice Indicator


**Blue dye contained in the product penetrates the root canal orifices and dyes them blue which makes it easier to find the main root canal and detect the cracks.**

### How to use

1. Remove dentine where the root canal orifices are expected. (Picture1)
2. Insert 1-3 drops into the bottom of the pulp cavity, wait 5-10 seconds. (Picture2)
3. Rinse and dry the pulp cavity. (Picture3)
4. The areas which show up in blue indicate the root canal orifices. (Picture4)
5. After use pull back the plunger of the syringe to prevent overflow of product, remove applicator, close the syringe with white cap.

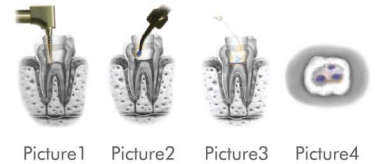

### Application

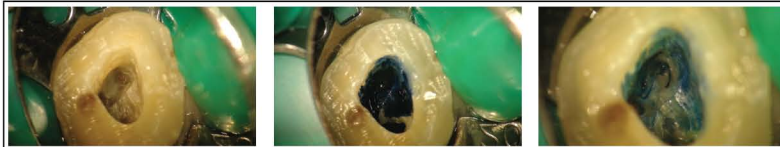

### Contents

#507 Canal-Seek 1.2ml\*3yr.  
/ Disposable tips

