Wireless Intraoral Camera

 **Air Direct**

# Dr's Cam Air

Enjoy wireless

Wireless type

User manual

Eng

Kor

CE

Wireless Intraoral Camera

 **Air Direct**

# Dr's Cam Air

Enjoy wireless

Wireless type

User manual

Eng

CE

|                               |           |
|-------------------------------|-----------|
| <b>1. Introduction</b>        | <b>04</b> |
| - Features                    |           |
| - Description                 |           |
| - Overview                    |           |
| <b>2. Safety Information</b>  | <b>05</b> |
| - Symbols                     |           |
| - Notes and reference         |           |
| - Installation and Connection |           |
| - Operation                   |           |
| - Maintenance                 |           |
| <b>3. List of Components</b>  | <b>09</b> |
| <b>4. Parts and functions</b> | <b>10</b> |
| <b>5. Installation</b>        | <b>11</b> |
| <b>6. How to use</b>          | <b>15</b> |
| <b>7. Maintenance</b>         | <b>17</b> |
| <b>8. Trouble Shooting</b>    | <b>18</b> |
| <b>9. Specification</b>       | <b>19</b> |
| <b>10. Accessories</b>        | <b>20</b> |
| <b>11. Warranty</b>           | <b>21</b> |
| <b>12. Warranty card</b>      | <b>22</b> |

# 1 Introduction

Thank you for purchasing the device. The instructions described below have been designed to provide the necessary information for the successful operation, daily maintenance and safety for the use of this unit.

## Features

---

- High speed digital wireless
- Autofocus
- Gyro mouse built-in
- 280° twist head
- Adjustable LED brightness
- Software compatibility
- Easy to read OLED screen
- Dental image software included (DrsViewer2)
- 3 color impressions

## Description


---

This device is an intraoral camera used to take images of the inside of the mouth or oral cavity.


## Overview

---

### 1) Camera

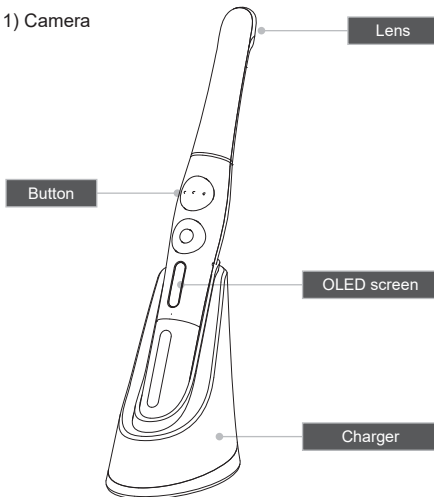### 2) Receiver part
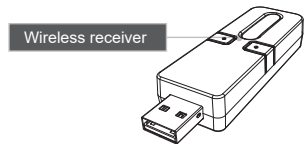
## 2 Safety information

It is important to understand the following information to use this product safely.

### Symbols

---

Warning and Caution used in this manual are shown below.

Indicates a potentially hazardous situation which, if not avoided, may result in serious injury or death.

Indicates a potentially hazardous situation which, if not avoided, may result in minor/moderate injury or in damage of property.

### Notes and References

---

Notes and References used in this manual are shown below.

Indicates that potential malfunctions may occur if the description is ignored.

Supplementary information to facilitate better operation.

## 2 Safety information

### Installation and Connection

#### Warning

- The adapter should be disconnected from the charger before connecting the power cable to prevent an electric shock.
- Keep away from sources of ignition to avoid fire or explosion.
- Protect the device from any liquid to avoid electric shock or functional error.
- Place the charger on a flat and stable surface. Do not drop device and charger to avoid internal defect that may cause overheating or fire.
- Do not place the electrical cord near a gas burner or sources of heat. Should the cord catch on fire, replace with the new one to avoid short-circuit.

#### Caution

- Do not use or leave the unit in areas of high temperature or exposed to direct sunlight, internal circuit defect may cause overheat or fire.
- Securely connect the dust-free power plug to the outlet.
- Use this device in well ventilated areas. Overheating may cause fire.
- Do not touch the terminal areas of charger of battery with fingers or conductive objects, such as metal wire or safety-pin. It may electric shock or fire.
- Do not use any charger other than supplied. It may cause fire due to overheating.
- Do not place barrier between camera and receiver. It would make hesitation wireless data translation.

## 2 Safety information

### Operation

#### Warning

- Do not plug the power cord with wet hands. This may result in an electric shock.
- In case of fluid leak from battery or partial discoloration/deformation of the device, immediately discontinue use.
- In case of partial discoloration of the device, immediately disconnect the power plug and repair it.
- In case battery fluid contacts the eye, immediately flush with copious amount of clean water and seek medical attention.
- Do not use this product on a person with a pacemaker as it may cause malfunction of the pacemaker.
- Should use disposable cover to avoid cross-infection.

#### Caution

- Disconnect the power cord by grasping the power plug to avoid injuries or burns.
- Avoid damage or use of unnecessary force on the power cord and device. This may cause an electric shock, fire, or malfunction.
- Should battery fluid contact the skin or clothes, immediately wash it off with copious amount water to avoid skin damage.
- Do not use this product for any purpose other than dental use.
- This device is not water prove. Use disposable cover when use to avoid wet the device. In case of malfunction due to this case, it is not covered by warranty.
- For safety, disconnect the adapter from charger in case of device is not charging.

## 2 Safety information

### Maintenance

---

#### Warning

- Use only accessories supplied to avoid fluid leak, explosion, or malfunction.
- Do not disassemble or overhaul the device. Unusual performance may cause injuries, electric shock, fire, or malfunction.
- Ensure that charger and camera are always separated from the device prior to cleaning.
- Be careful from any liquid when clearing.


#### Caution

- Do not use solvent (containing orange oil, ethanol or acetone) when clean the device. It may cause malfunction.
- User must be in charge of operation, maintenance and check.


### 3 List of components


Please check the contents described below when removing the contents from the original packaging.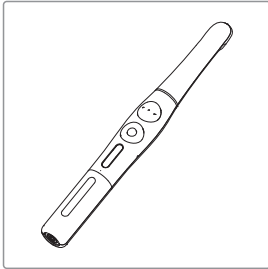


Camera  
(Part No. DCAMM1001)

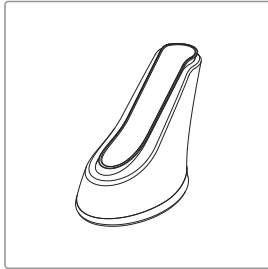
Charger  
(Part No. DCAMM2001)

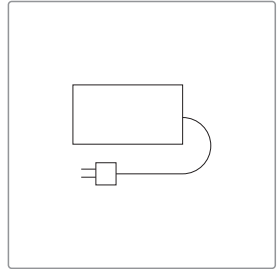
Adapter  
(Part No. AC0105)

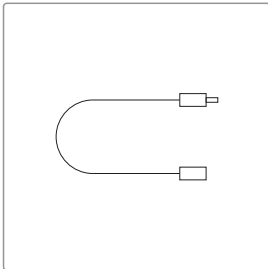Power cable  
(Part No. AC0111)

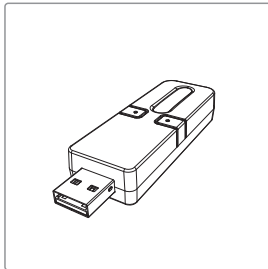
Receiver-Air Direct  
(Part No. AC0216)

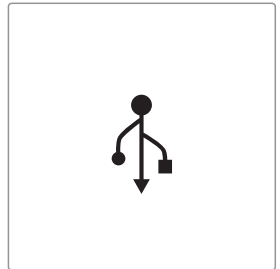Data USB  
(Part No. AC0201)


Disposable plastic cover x 2ea  
(35 x 100mm / narrow)  
(Approx. 100pcs)  
(Part No. AC0103)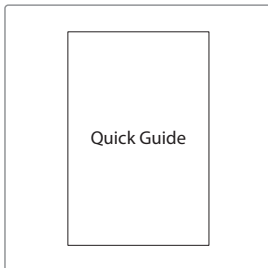

Quick Guide  
(Part No. DCAM080801)

## 4 Parts and functions
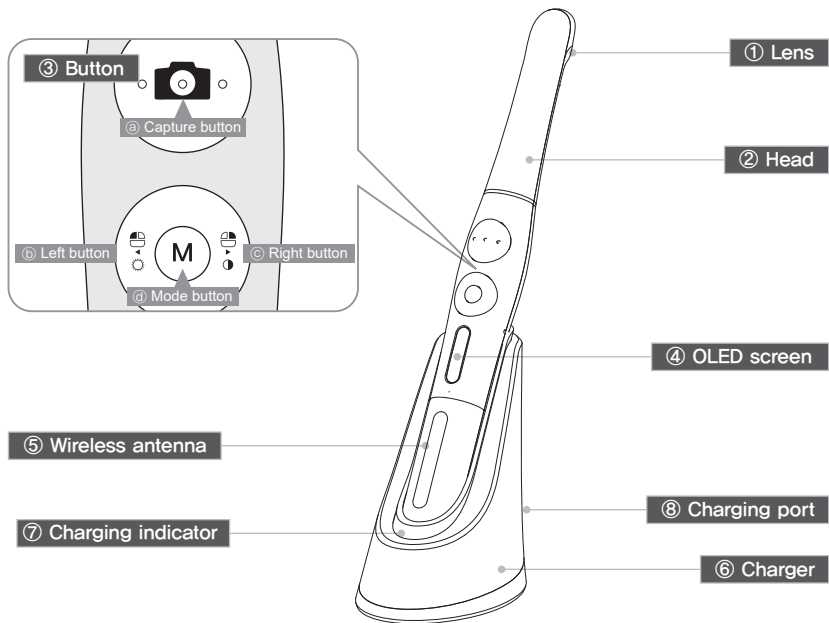
### ① Lens

- Image is reflected on the lens through prism.

### ② Head

- Twistable 280° degrees

### ③ Button

- ⓐ Capture button
  - Image capture, focus adjusting, select
- ⓑ Left button
  - Mouse left button
- ⓒ Right button
  - Mouse right button
- ⓓ Mode button
  - Mouse moving activate, setup mode, cancel

### ④ OLED screen

- Show the device states

### ⑤ Wireless antenna

- Data is sent from this antenna to receiver.

### ⑥ Charger


- Charging the device

### ⑦ Charging indicator

- Red: On charging  
- Blue: Full charged

### ⑧ Charging port


- Connect the AC adapter for charging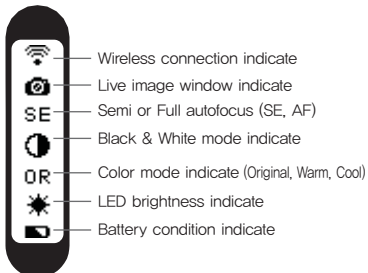

## 5 Installation

### 1. Connection between adapter and charger

- 1) Securely insert the power cable with 100~240V power outlet.
- 2) Connect power cable with adapter.
- 3) Connect adapter with charger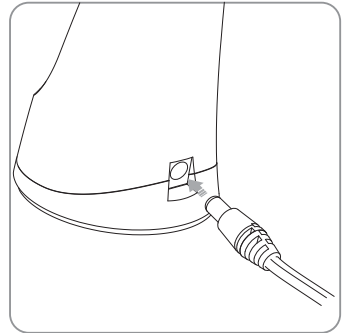


Note

- Use only the original adapter supplied.
- When the adapter is connected with charger, red and blue light will blink.
- If red and blue light is not blinking when adapter is connected, it is a charger malfunction.

### 2. Charging

- Insert the handpiece into the vertical slot at the front of the charger.
- Red light on : Charging in progress
- Blue light on : Charging process is complete

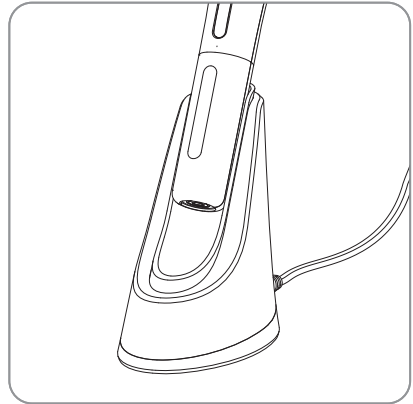Note

- If the remaining battery is extremely low, red light may flicker while recharging, which is not a malfunction.


Reference

- It will take approximately 1 hours 40 minutes before the battery will be fully charged.

# 5 Installation

## 3. Receiver installation


- 1) Connect the receiver to the USB port on your PC

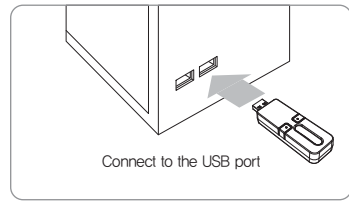- 2) After approximately 5 seconds, the receiver will display a logo and the receiver's default number "001"


- 3) To change to a number that does not overlap with other receivers, press the ▲ and ▼ buttons on the receiver to change the number. (Press and hold the ▲ and ▼ buttons to change quickly.)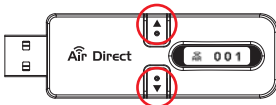

## 4. Camera connection


- 1) To use the camera for the first time or to connect to another receiver, the camera requires a receiver connection setup.
- 2) Press and hold the capture button to power on the camera.
- 3) Double-click the Capture button to search for a connectable receiver.

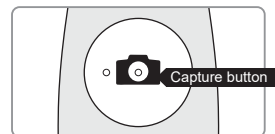- 4) When number of connectable receiver is displayed, use the left/right buttons on the camera to navigate to the receiver number to connect and press the capture button to select it.


Select the number of receiver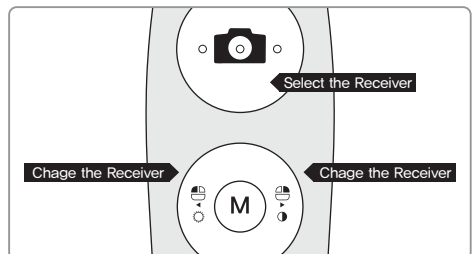


## 5 Installation


5) When "Air Direct connected" message shows up and connected well, receiver's number will display in the main screen which is connected to wireless icon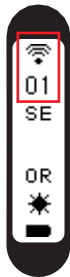

Successful connection

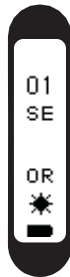Not connected with the receiver

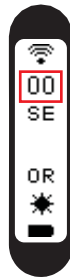
Number with at least 3 digits

Reference

More than 3 digits of number will display as "00" and you can check the whole number with pressing M button

6) when you press and hold on M button while the viewer is turned off, 3 digits number of connected receiver will display

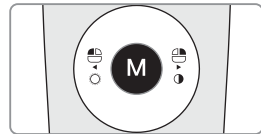Reference

- When the camera is powered off and on, it automatically connects to the receiver with the set number
- If it doesn't connect automatically after 20 seconds, you can connect manually by double-clicking the capture button

### 5. Dr's Viewer2 installation

- Download Dr's viewer if there is no any viewer software program

Reference


If you are using any other software, please contact your supplier

< Installation Dr's Viewer2 screen >

# 5 Installation

## 6. Camera device setup

- Click and hold left button and right button for 3 seconds. Then the device is turn to setup mode.

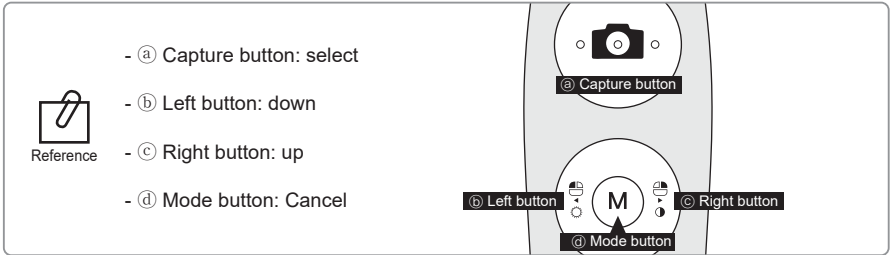### 1) Capture key setting

- Select appropriate capture key
- Default value is "Space"

Reference

Capture key can be changed with "HIDDRs Manager" in the CD.

### 2) Mouse speed

- Select appropriate gyro mouse speed. (5 levels)
- Default level is 3

### 3) Turn off time

- Select turn off time of device (5, 10, 15, 20, 30 mins)
- Default time is 5 mins.

### 4) Autofocus & Autocapture

- Select "Disable" or "Enable"
- Default value is "Disable"

To avoid shaking image there is Auto capture function.

Reference

\* Auto capture:

- Semi autofocus (Disable): Half click and hold the condition, image is captured after 1 seconds after get focus, and image is captured for every 2 seconds.
- Full autofocus (Enable): Half click is image capture and hold the condition, image is captured for every 2 seconds.


## 6 How to use

### 1. Turn on the device

- Click the capture button. Then the device turn on with LED blink with beep sound.

### 2. Image capture

- Click camera icon on the DrsViewer2. Then live image is shown.
- Half click is focus adjusting, and click is image capture.  
If the mode is full autofocus mode, half click is image capture.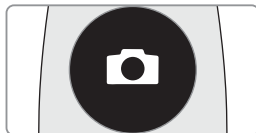


Reference

Default setup is semi autofocus. And capture key is "Space"  
To use other company software, need to check capture key.  
Or need to use twain driver.


### 3. Mouse

- To move mouse cursor click mode button and shake the device.  
Then mouse cursor is moved as device shaken.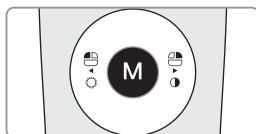



### 4. Color impression changing

- Click mode button and capture button on live image.  
Then the color impression is changed.
- Original ⇒ Warm ⇒ Cool color impression is rotated.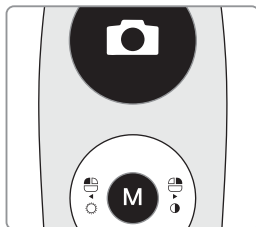

### 5. Brightness change

- Click mode button with mouse left button. Then LED brightness is changed.

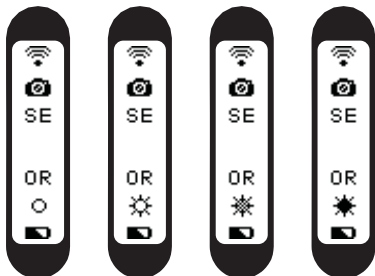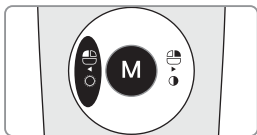Turn off ⇒ Brightness LV1 ⇒ Brightness LV2 ⇒ Brightness LV3 is rotated.



Reference

Brightness change is worked on live image is on.

## 6 How to use

### 6. Black & White mode

- Click mode button with mouse right button.  
Then LED turn off, and live image changed to black & white mode.

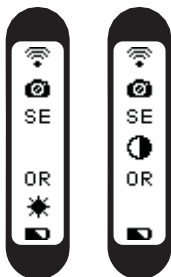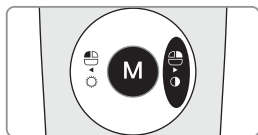Normal ⇒ Black & White mode is rotated.

Reference

Black & White mode is worked on live image is on.


### 7. Setup mode

- Click mouse left button and mouse right button both for 3 seconds.  
Then setup mode is shown.
- About setup mode, please refer,  
"5. Installation ⇒ 6. Camera device setup."

### 8. Turn off

- Click the capture button for 3 seconds, the device is turned off.
- The device is turned off automatically, if there is no signal for minutes set on setup mode.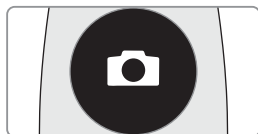


Reference

- User can adjust the sleep mode time for 5, 10, 15, 20, 25, and 30 minutes on setup mode.
- If wireless connection is not connected, device will be turn off after 2 minute.

### 9. Reset

- If the device doesn't work normally, reset the device. Then the device will be reboot.

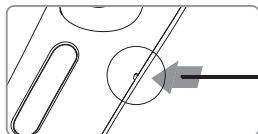## **Cleaning and disinfection**

---

- Clean the camera and charger gently with clean, soft, lint-free cloth.
- Avoid using solvents which contain ethanol or acetone.
- Do not use sharp or pointed tools to clean as they may damage or scratch the plastic.
- Be careful not to spill any liquid into the device.
- Use disposable cover when use the device to avoid cross-infection and spilling any liquid into the device.

## **Maintenance and replacement of the battery.**

---

- Do not disassemble the device for any purpose.
- To replace the battery, please contact authorized dealer or manufacturer.
- It is recommended for rechargeable battery to be charged immediately before use to maximize efficiency.

## 8 Troubleshooting

If the device does not work normally, please refer to the following points.

| Problem                      | Cause and solution  |
|------------------------------|---|
| Button doesn't work          | <p>If the button is pressed with excessive force, it will not work, try pressing the button gently. And check the button condition.</p> <p>If the device is not recognized well, the button could not work well. Refer "5. Installation", "6. How to use"</p> <p>If connection between camera and receiver is not connected, the button doesn't work. Please avoid any barrier between camera and receiver.</p>   |
| Display screen does not work | <p>If the battery voltage is extremely low, the device is not turned on. Please charge the battery.</p>   |
| Power off while in operation | <p>If the battery voltage is extremely low, the device could be turned off. Please charge the battery.</p>  |
| Not able to capture image    | <p>Capture key-value would not match with other company dental viewer software. Please check keyboard short-cut. If you don't know it please ask the dental viewer software vendor.</p> <p>And check the key-value of device. Or use twain driver included.</p>   |
| Live image is not shown      | <p>If the device is not recognized well, live image is not shown. Please check camera and receiver connection, and check imaging device of device manager.</p> <p>If there is TV tuner in the PC. The image could be not shown. Check "Disable" the TV tuner device on the device manager.</p> <p>If you use other company dental viewer software, it would be related with its setting. Please ask to the dental viewer software vendor.</p> <p>If you use other company dental viewer software, other imaging device (ex. Web-cam, etc.) should be checked for "Disable".</p> <p>This device support only MJPG (MPEG) format. If you use other company dental viewer software, please check the viewer support MJPG or MPEG. Please ask to the dental viewer software vendor.</p> <p>Some codec makes block or delay the live image shown. Uninstall the codec file. (ex. klite codec and etc.)</p> |

\* If the trouble is not fixed after following solution, please contact your local dealer or manufacturer.

# 9 Specification

## ■ Equipment Class

|                                     |                           |
|-------------------------------------|---------------------------|
| Model:                              | IC-DCAM70                 |
| Trademark:                          | Dr's Cam Air              |
| Power supply:                       | 3.7V Lithium Ion          |
| Receiver:                           | 5V USB power              |
| Protection from ingress of liquids: | IPX0 (ordinary equipment) |

## ■ Operation, transport, and storage condition for the main unit

|                      | Operation      | Storage | Transport |
|----------------------|----------------|---------|-----------|
| Temperature          | 10~30C         | 10~40C  | -10~50C   |
| Humidity             | 10~80%         | 20~80%  | 20~80%    |
| Atmospheric pressure | 800 ~ 1360 hPa |         |           |

## ■ Additional Information

|                  |                   |                                 |
|------------------|-------------------|---------------------------------|
| Camera           | Operation voltage | 3.7V                            |
|                  | Measurement       | Approx. 255 x 30 x 28 mm        |
|                  | Weight            | Approx. 89g                     |
|                  | Resolution        | 5M pixels                       |
|                  | Sensor/Lens type  | CMOS / Liquid lens              |
| Charger          | Measurement       | Approx. 110 x 105 x 68 mm       |
|                  | Weight            | Approx. 100 g                   |
|                  | Operation voltage | 5V                              |
| Disposable cover | Quantity          | Approx. 100 pcs by weight x 2ea |
|                  | Raw material      | Polyethylene                    |
| Adapter          | Input voltage     | AC 100 ~ 240 V                  |
|                  | Output voltage    | DC 5V 2A                        |
|                  | UL number         | E333054                         |

## ■ Minimum system requirement

|            |                      |
|------------|----------------------|
| PC system: | 3.3 Ghz CPU, 4GB RAM |
| OS:        | Windows 7            |
| USB port:  | USB 2.0              |

(Internet access would need for some kinds of PC environments)

\* Upper specification is changeable without prior notice.

## ■ Symbol

This device conforms with the European Directive, 93 / 42 / EEC which includes the requirements for electromagnetic compatibility

This symbol is appixed to fulfill the requirements of EU Directive 2012/19/EU Article 11. This equipment cannot be disposed of as unsorted municipal waste within the European Union. Follow local regulations for disposal.

SN

Serial number 00 00 00 0000  
Model / Month / Year / Serial


Warning, Caution


## 9 Specification

|   |   |
|---|---|
|  Type B applied part                                 |  Manufacturer                  |
|  Authorized representative in the European Community |  Check user manual.            |
|  Prevent by wet.                                     |  This box is paper recyclable. |
|  Transport temperature.                              |  No hooks                      |
|  Reuse is not allowed                                |  Fragile item                  |
|  Do not device on place where get direct sunshine.   |   |


## 10 Accessories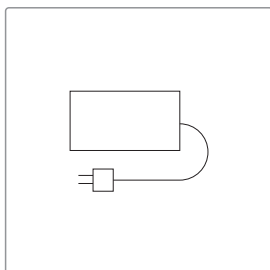

Adapter  
(Part No. AC0105)

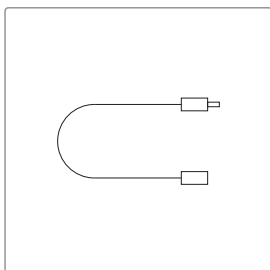
Power cable  
(Part No. AC0111)

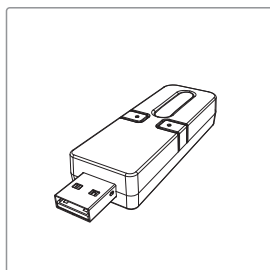
Receiver-Air Direct  
(Part No. AC0216)

Disposable plastic cover  
(35 x 100mm / narrow)  
(Approx. 100pcs)  
(Part No. AC0103)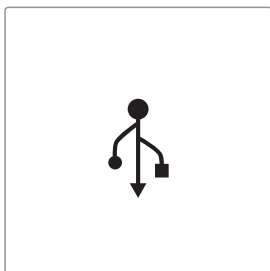

Data USB  
(Part No. AC0201)

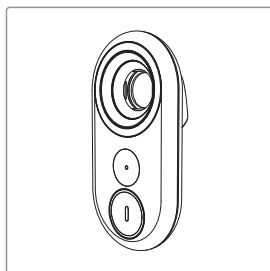EIOS  
(Part No. AC0213)


# 11 Warranty

This product is only shipped after strict inspection. Should the product incur any problems under normal use conditions within one year from the date of purchase, the repair will be free of charge.

However, repair of consumables (Disposable Cover, and etc.) are not covered by the warranty.

For more information, refer to the 12. Warranty card attached.

Serial number check.

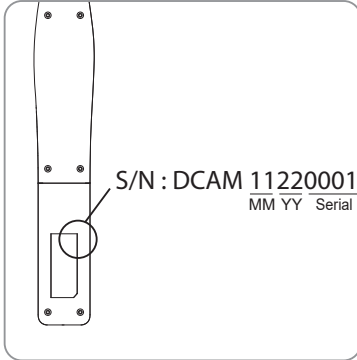**EC REP Good Doctors Germany GmbH**

Gerhard-Domagk-Str. 2, 53121 Bonn, Germany  
TEL: +49 (0) 228/53 44 14 65 FAX: +49 (0) 228/53 44 14 66  
[www.gooddrs.de](http://www.gooddrs.de)

**Good Doctors Co., Ltd.**

#208, B-dong, 283 Bupyeong-daero, Bupyeong-gu  
(Woolim Lions Valley, Cheongchoen-dong), Incheon,  
21315, Republic of Korea  
Tel : + 82 32 424 6325 Fax : + 82 32 424 6326  
E-mail : [info@gooddrs.com](mailto:info@gooddrs.com) Website : [www.gooddrs.com](http://www.gooddrs.com)

We guarantee the quality of the product as follows

**Name of product :**

**Name of model :**

**Customer information**

- Office name :
- Office phone :

**Distributor**

- Dealer name :

**Assurance period**

- Date of purchase : from \_\_\_\_\_ , 20\_\_\_\_
- Product is for 1 year except accessories.

**< Assurance details >**

1. If there is any defect of manufacturer or product has any fault itself within the assurance period, it will be replaced with the same product.
  - Malfunction and damage according to user's carelessness.
  - Malfunction and damage according to natural disaster such as fire, earthquake and flood etc.
  - Malfunction and damage according to user's intention or accident.
  - Malfunction and damage according to using incompatible product.
3. The manufacturer does not accept any liability resulting misuse.

**Good Doctors Co., Ltd.**  
 #208, B-dong, 283 Bupyeong-daero, Bupyeong-gu (Woolim Lions Valley, Cheongchoen-dong), Incheon, 21315, Republic of Korea  
 Tel: +82 32 424 6325 Fax: +82 32 424 6326

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